

SpectraGel[®] 2Q

100% SILICONE GEL FOR SCAR REDUCTION THERAPY

PRODUCT INFORMATION DATA SHEET

DESCRIPTION: SpectraGel[®]2Q has the same indications for use, as the original SpectraGel[®] silicone elastomer gel (see reverse), however it has been specially formulated for sensitive skin and the topical management of erythema (redness) often associated with laser surgery and resurfacing of the face. Scars that have a raised and discolored appearance, may soften and fade with regular use of SpectraGel[®]2Q, as directed. This product is clear, odorless, and self-drying. SpectraGel[®]2Q has improved drying time than the original SpectraGel[®] formulation, and is also ideal for applying under makeup or concealer. When properly applied, SpectraGel[®]2Q instantly bonds to the affected scar, forming a protective coating over the surface of the scar and stays on until it is washed off with soap and warm water.

SpectraGel[®]2Q is available in a 10gr. tube (MFG PN 300-310)
The 10 gram tube is approximately a 90 day supply for a 3-4 inch scar.

WARRANTY INFORMATION: Spectrum Designs, Inc. warrants that reasonable care was used in the manufacture of this product. Spectrum Designs, Inc. does not warrant either a good effect or against an ill effect following this product's use. Spectrum Designs, Inc.'s sole responsibility in the event that Spectrum determines a product is defective, when shipped by Spectrum, shall be replacement of the product. This warranty is in lieu of and excludes all other warranties, not expressly herein, whether expressed or implied, by operating of law or otherwise including any implied warranties of merchantability or fitness for product use.

INSTRUCTIONS FOR USE: Keep out of reach of children. For external use only. Do not apply this product to broken skin, open wounds, third degree burns, mouth, eyes, or mucous membranes. If irritation or redness develops, discontinue use and consult your physician.

- Step 1.** Wash both hands and the scar with a mild soap and water, and pat dry.
- Step 2.** Apply a very small amount of SpectraGel[®]2Q, directly onto the scar with fingertips.
- Step 3.** In a circular motion, working from the center of the scar outward, cover the scar tissue with a thin layer of SpectraGel[®]2Q.
- Step 4.** If tackiness is observed, wipe away any excess gel.
- Step 5.** Allow SpectraGel[®]2Q to dry for about 3-5 minutes to avoid staining of clothing.
- Step 6.** Apply SpectraGel[®]2Q every 12 hours, and follow the above steps 1-5 with each application. Continue use for 8-12 weeks, or as directed by your physician.



MANUFACTURED BY:

SPECTRUM
DESIGNS, INC.

132 Easy St., Unit D5, Buellton, CA 93427 USA
www.spectragel.com

PID PN 300030
ECR REV. 09-2017-001